DENTAL UNIVERSAL PROSTHETIC PRIMER

CLEARFIL™ CERAMIC PRIMER PLUS

CE
0197

ENGLISH INSTRUCTIONS FOR USE

I. INTRODUCTION

CLEARFIL CERAMIC PRIMER PLUS is a dental universal prosthetic primer that provides an enhanced adhesive surface to ceramic (e.g. conventional porcelains, lithium disilicate, zirconia), hybrid ceramics (e.g. ESTENIA C&B), composite resin and metal.

II. INDICATIONS

CLEARFIL CERAMIC PRIMER PLUS is indicated for the following uses:
[1] Surface treatment of prosthetic restorations made of ceramic, hybrid ceramics, composite resin or metal

[2] Intraoral repairs of fractured restorations made of ceramics, hybrid ceramics, composite resin or metal

III. CONTRAINDICATIONS

Patients known to have hypersensitivity to methacrylate monomers

IV. POSSIBLE SIDE EFFECTS

The side effect members may turn white when contacted by the product due to the coagulation of proteins. This is usually a temporary phenomenon that will disappear in a few days. Instruct patients to avoid irritating the affected area while brushing.

V. PRECAUTIONS

1. This product contains substances that may cause allergic reactions. Avoid use of the product in patients with known allergies to methacrylates or other common allergens.

2. If the patient demonstrates a hypersensitivity reaction, such as rash, eczema, feelings of irritation, ulcer, swelling, itching, or numbness, discontinue use immediately.

3. Avoid direct contact with the skin and/or soft tissue to prevent hypersensitivity. Wear gloves or take appropriate precautions when using the product.

4. Exercise caution to prevent the product from coming in contact with the skin or getting into the eye. Before using the product, cover the patient's eyes with a cloth and/or glasses to protect them in the event of splashing materials.

5. If the product comes in contact with human body tissues, take the following steps:
-If the product gets in the eyes: Immediately wash the eye with copious amounts of water and consult a physician.
-If the product comes in contact with the skin or the oral mucosa: Immediately wipe the area with a clean cloth or a gauze pad moistened with water and rinse the area with water.

6. Exercise caution to prevent the patient from accidentally swallowing the product. If swallowed, instruct the patient to vomit.

7. Do not reuse the Applicator brush to prevent cross-contamination. Applicator brush is single-use only. Discard it after use.

2. Handling and manipulation precautions

1. The product must not be used for any purposes other than specified in [II. INDICATIONS].

2. The product is restricted to dental professionals.

3. Do not mix the product with bonding agents or use in conjunction with other products, combining materials may cause a change in physical properties, resulting in a loss of strength.

4. The product should be used shortly after dispensing. It contains volatile ethanol. As the solvent evaporates, the viscosity increases and may result in a loss of strength.

5. Perform bonding (or cementation) soon after treating the restoration surface with the product.

6. If the product is accidentally contaminated with saliva or blood, wash it with water, dry, clean with an etching agent (e.g. K-ETCHANT Syringe), and re-treat.

7. If the product must be stored at 2°C-36-46°F when not in use, and needs to be brought to room temperature for 15 minutes before using, otherwise, excess liquid may be dispensed or ooze after use.

3. Product must be kept away from extreme heat, direct sunlight or a flame.

4. The bottle cap should be replaced as soon as the liquid has been dispensed from the bottle. This prevents evaporation of volatile ingredients.

5. The product must be stored in a proper place where only dental practitioners can access.

VI. COMPONENTS

Please see the inside of the package for contents and quantity.

•Principals ingrediént

•3-Methacryloyloxypropyl trimetoxiláno

•10-Metacryloyloxydecyl dihydrogenphosphate (MDP)

•Etilanol

VII. CLINICAL PROCEDURES

A. Surface treatment of prosthetic restorations made of ceramic, hybrid ceramics, composite resin or metal

1. Place a rubber dam to accomplish an appropriate isolation for adhesive procedures.

2. If necessary, place a rubber dam to accomplish an appropriate isolation for adhesive procedures.

3. Treatment of the mucous buccal to prevent the coagulation of proteins. This is an eventuality temporary who disrupts habitually in few quarters. Demands that aux patients d'interer toute initiation lors du brossage de la zone interférente.

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