**GC Fuji TRIAGE® CAPSULE**

RADIOPAQUE GLASS IONOMER PROTECTION AND TEMPORARY RESTORATIVE MATERIAL IN CAPSULES

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- RECOMMENDED INDICATIONS**
- Fissure protection.
  - Root surface protection.
  - Hypersensitivity prevention and control.
  - Protection for immature enamel.
  - Temporary filling of endodontic access.
  - Intermediate restorations.

**CONTRAINDICATIONS**

1. Pulp capping.  
2. In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

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Powder / Liquid Ratio (g / g)	0.30 / 0.15
Mixing Time (sec.)	10"
Working Time (min., sec.)	1'40"
Net Setting Time (min., sec.)	2'30"
Final Finishing Commencing Time	6'00"
Final Finishing Commencing Time if light cured (PINK Command set)	4'00"

Test conditions : Temperature (23+/1°C), Relative humidity (50+/-5%)

**A. FISSURE PROTECTION****ROOT SURFACE PROTECTION****HYPERSENSITIVITY PREVENTION AND CONTROL****PROTECTION FOR IMMATURE ENAMEL****1. Preparation of the tooth surfaces (e.g. for fissure protection or root surface protection)**

- a) After cleaning the tooth surfaces (prophylaxis with pumice and water) in usual manner, rinse thoroughly with water. Avoid aggravating the operculum.  
Note:  
If extra retention is desired, application of GC CAVITY CONDITIONER (10 seconds) or GC DENTIN CONDITIONER (20 seconds) is recommended.

- b) Dry by blotting with a cotton pellet or gently blowing with an air syringe (Fig. A-1). DO NOT DESICcate. Best results are obtained when prepared surfaces appear moist (glistening).

**2. Mixing**

- a) Before activation, shake the capsule or tap its side 2 or 3 times on a hard surface to loosen the powder (Fig. A-2).

- b) To activate the capsule, push the plunger until it is flush with the main body (Fig. A-3).

- c) Immediately place the capsule into a GC Capsule Applier and click the lever once (Fig. A-4). The capsule is now activated.

Note :

The capsule should be activated just before mixing and used immediately.

- d) Immediately remove the capsule from the applier and set the capsule into a capsule mixer or amalgamator. Mix for 10 seconds at high speed (approximately 4,000 RPM) (Fig. A-5).

**3. Placement**

- a) Immediately remove the mixed capsule from the mixer and load it into the GC Capsule Applier.

- b) Make two clicks to prime the capsule then syringe (Fig. A-6). The working time is 1 minute 40 seconds from the start of mixing at 23°(73.4°F). Higher temperatures will shorten working time.

- c) Extrude the mixture onto the tooth surface (Fig. A-7) then use a brush to spread a thin film of GC Fuji TRIAGE® directly over the root surface or hypersensitive area or over the occlusal surface and into the pits and fissures.

Note :

- If a faster set is desired, use a visible light curing device\* for 20-40 seconds. Place light source as closely as possible to the cement surface. This function applies only to the Pink Shade. After light cure, it is advisable to protect the surface with a varnish.

- 2) To adjust the direction of the nozzle, hold the applier with the capsule towards you and turn the capsule body.

- 3) To remove the used capsule, push the applier release button. Twist the capsule and pull upwards.

- d) After placement, when the material starts to lose the glossy appearance (or after curing with the light curing device), apply GC Fuji VARNISH (blow dry) or GC Fuji COAT LC (light cure) to the sealed area and the margins using a cotton pellet or sponge (Fig. A-8).

- e) Finishing under air/water spray can be performed 6 minutes from start of mix (chemically set) or 4 minutes if light cured. Use a supradent diamond bur or a silicone finishing point (Fig. A-9).

- f) Apply GC Fuji VARNISH or GC Fuji COAT LC to the area again (Fig.A-10).

**B. TEMPORARY FILLING OF ENDODONTIC ACCESS****1. Cleaning the pulp chamber**

- a) After appropriate pulp treatment, clean and gently dry the pulp chamber with an air syringe (Fig. B-1).

- b) Fill the chamber with a cotton pellet (Fig. B-2).

**2. Mixing**

See directions in A (above), Section 2.

**3. Placement**

- a) See directions in A, Section 3, a and b.

- b) Extrude the mixture directly over the cotton pellet (Fig. B-3).

Note :

- If a faster set is desired, use a visible light curing device\* for 20-40 seconds. Place light source as closely as possible to the cement surface. This function applies only to the Pink Shade. After light cure, it is advisable to protect the surface with a varnish.

c) Moisture protection and Finishing

See directions in A, Section 3, d, e and f.

**C. INTERMEDIATE RESTORATIONS****1. Caries removal**

- a) Remove caries and loose debris with hand instruments.

- b) For better retention, it is recommended to gently clean the carious surface with GC CAVITY CONDITIONER for 10 seconds or GC DENTIN CONDITIONER for 20 seconds.

- c) Rinse thoroughly with water. Dry by blotting with a cotton pellet or gently blowing with an air syringe (Fig. C-1). DO NOT DESICCATE. Best results are obtained when prepared surfaces appear moist (glistening).

**2. Mixing**

See directions in A (above), Section 2.

**3. Placement**

- a) See directions in A, Section 3, a and b.

- b) Extrude the mixture directly over the prepared lesion or tooth surface (Fig. C-2). Form the contour and if possible cover with a matrix.

Note :

- If a faster set is desired, use a visible light curing device\* for 20-40 seconds. Place light source as closely as possible to the cement surface. This function applies only to the Pink Shade. After light cure, it is advisable to protect the surface with a varnish.

c) See directions in A, Section 3,d,e and f.

\* NOTE :

The initial set of GC Fuji TRIAGE® PINK can be accelerated using the energy from a dental halogen light curing device.

**SHADE**

Pink, White

**STORAGE**

Recommended for optimal performance, store at temperature of 4-25°C (39.2-77.0°F)

(Shelf life : 2 years from date of manufacture)

**PACKAGE**

50 capsules

Average contents per capsule : 0.30g powder and 0.15g (0.12mL) liquid

Minimum net volume of mixed cement per capsule : 0.13mL

**CAUTION**

1. In case of contact with oral tissue or skin, remove immediately with a sponge or cotton soaked in alcohol. Flush with water.

2. In case of contact with eyes, flush immediately with water and seek medical attention.

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Pulver / Flüssigkeit (g / g)	0.30 / 0.15
Mixing Time (sec.)	10"
Working Time (min., sec.)	1'40"
Net Setting Time (min., sec.)	2'30"
Final Finishing Commencing Time	6'00"
Final Finishing Commencing Time if light cured (PINK Command set)	4'00"

Test conditions : Temperature (23+/1°C), Relative humidity (50+/-5%)

**A. FISSURE PROTECTION****ROOT SURFACE PROTECTION****HYPERSENSITIVITY PREVENTION AND CONTROL****PROTECTION FOR IMMATURE ENAMEL****1. Preparation of the tooth surfaces (e.g. for fissure protection or root surface protection)**

- a) After cleaning the tooth surfaces (prophylaxis with pumice and water) in usual manner, rinse thoroughly with water. Avoid aggravating the operculum.  
Note:  
If extra retention is desired, application of GC CAVITY CONDITIONER (10 seconds) or GC DENTIN CONDITIONER (20 seconds) is recommended.

- b) Dry by blotting with a cotton pellet or gently blowing with an air syringe (Fig. A-1). DO NOT DESICcate. Best results are obtained when prepared surfaces appear moist (glistening).

**2. Mixing**

- a) Before activation, shake the capsule or tap its side 2 or 3 times on a hard surface to loosen the powder (Fig. A-2).

- b) To activate the capsule, push the plunger until it is flush with the main body (Fig. A-3).

- c) Immediately place the capsule into a GC Capsule Applier and click the lever once (Fig. A-4). The capsule is now activated.

Note :

The capsule should be activated just before mixing and used immediately.

- d) Immediately remove the capsule from the applier and set the capsule into a capsule mixer or amalgamator. Mix for 10 seconds at high speed (approximately 4,000 RPM) (Fig. A-5).

**3. Placement**

- a) Immediately remove the mixed capsule from the mixer and load it into the GC Capsule Applier.

- b) Make two clicks to prime the capsule then syringe (Fig. A-6). The working time is 1 minute 40 seconds from the start of mixing at 23°(73.4°F). Higher temperatures will shorten working time.

- c) Extrude the mixture onto the tooth surface (Fig. A-7) then use a brush to spread a thin film of GC Fuji TRIAGE® directly over the root surface or hypersensitive area or over the occlusal surface and into the pits and fissures.

Note :

- If a faster set is desired, use a visible light curing device\* for 20-40 seconds. Place light source as closely as possible to the cement surface. This function applies only to the Pink Shade. After light cure, it is advisable to protect the surface with a varnish.

- 2) To adjust the direction of the nozzle, hold the applier with the capsule towards you and turn the capsule body.

- 3) To remove the used capsule, push the applier release button. Twist the capsule and pull upwards.

- d) After placement, when the material starts to lose the glossy appearance (or after curing with the light curing device), apply GC Fuji VARNISH (blow dry) or GC Fuji COAT LC (light cure) to the sealed area and the margins using a cotton pellet or sponge (Fig. A-8).

- e) Finishing under air/water spray can be performed 6 minutes from start of mix (chemically set) or 4 minutes if light cured. Use a supradent diamond bur or a silicone finishing point (Fig. A-9).

- f) Apply GC Fuji VARNISH or GC Fuji COAT LC to the area again (Fig.A-10).

**B. TEMPORARY FILLING OF ENDODONTIC ACCESS****1. Cleaning the pulp chamber**

- a) After appropriate pulp treatment, clean and gently dry the pulp chamber with an air syringe (Fig. B-1).

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**GC Fuji TRIAGE® CAPSULE**

RADIOPAK GLASS IONOMER PROTECTION AND TEMPORARY RESTORATIVE MATERIAL IN CAPSULES

For use only by a dental professional in the recommended indications.

**RECOMMENDED INDICATIONS**

1. Fissure protection
2. Root surface protection
3. Hypersensitivity prevention and control
4. Bonding to immature enamel
5. Temporary filling of endodontic access.
6. Intermediate restorations.

**CONTRAINDICATIONS**

1. Pure capping.  
2. If any part of the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

**DIRECTIONS FOR USE**

PINK Shade is command set, WHITE Shade is chemical set only.

Powder / Liquid Ratio (g / g)  
0.30 / 0.15  
Mixing Time (sec.)  
10\*  
Working Time (min. sec.)  
1'40"  
Net Setting Time (min. sec.)  
2'30"  
Final Finishing Commencement Time  
6'00"  
Final Finishing Time if light cured (PINK Command set)  
4'00"  
Test conditions : Temperature : (23±1°C),  
Relative humidity : 50% (±5%)

A FISSURE PROTECTION  
ROOT SURFACE PROTECTION  
HYPERSENSITIVITY PREVENTION AND CONTROL  
PROTECTION FOR IMMATURE ENAMEL

1. Preparation of the tooth surfaces (e.g. for fissure protection or root dentine protection).
2. After rinsing the tooth surface (prophyaxis with pumice and water) in usual manner, rinse thoroughly with water. Avoid agitating the preparation.
3. If extra retention is desired, application of GC CAVITY CONDITIONER (10 seconds) or GC DENTIN CONDITIONER (20 seconds) is recommended.

b) Dry by blowing with a cotton pellet or gently blowing with an air syringe (Fig. A-1). DO NOT DESICATE. Best results are obtained when prepared surfaces appear moist (glistening).

2. Mixing:  
a) Before activation, shake the capsule or tap it side 2 or 3 times on a hard surface to loosen the powder (Fig. A-2).

b) To open the capsule, push the plunger until it is flush with the main body (Fig. A-3).

c) Immediately place the capsule into a GC Capsule Applier and click the lever once (Fig. A-4). The capsule is now activated.

Note : The capsule should be activated just before mixing and used immediately.

d) Immediately transfer the capsule from the applier and set the capsule into a capsule mixer or amalgamator. Mix for 10 seconds at high speed (approximately 4,000 RPM) (Fig. A-5).

3. Placement:  
a) Immediately remove the mixed capsule from the mixer and load it into the GC Capsule Applier.

b) Apply pressure to prime the capsule then (Fig. A-6).

c) Extrude the mixture onto the tooth surface (Fig. A-7) then with a brush to spread a thin film of GC Fuji TRIAGE® directly over the root surface or hypersensitive areas or over the occlusal surface and into the pits and fissures.

Note : If a faster set is desired, use a visible light curing device\* for 20-40 seconds. Place light source as closely as possible to the fissure or dentine surface. If the light source is too far from the tooth, it is advisable to protect the surface with a varnish.

2) Adjust the direction of the nozzle, hold the applier with the capsule towards you and turn the capsule body.

3) To remove the used capsule, push the applier release button. Turn the capsule around 90 degrees.

a) After rinsing the tooth surface, repeat the procedure with a new capsule (Fig. A-8).

e) Finishing under air water spray can be performed 6 minutes from start of mix (chemically set) or 4 minutes after the placement of a soft-curing diamond bur or a silicone finishing point (Fig. A-9).

f) Apply GC Fuji VARNISH or GC Fuji COAT LC to the area again (Fig. A-10).

B. TEMPORARY FILLING OF ENDODONTIC ACCESS

1. Cleaning the pulp chamber.  
a) After appropriate pulp treatment, clean and gently dry the pulp chamber with an air syringe (Fig. B-1).

b) Fill the chamber with a cotton pellet (Fig. B-2).

2. Mixing:  
See directions in A (above), Section 2.

3. Placement:  
a) See directions in A, Section 3, a and b.

b) Extract the mixture directly over the cotton pellet (Fig. B-3).

Note : If a faster set is desired, use a visible light curing device\* for 20-40 seconds. Place light source as closely as possible to the cement surface. This function applies only to the Pink Shade. After light cure, it is advisable to protect the surface with a varnish.

c) Mixing and Light Curing:

See directions in A, Section 3, d and e.

C. INTERMEDIAL RESTORATION

1. Care is removed.

a) Remove caries and loose debris with hand instruments.

b) For better retention, it is recommended to gently clean the carious surface with GC CAVITY CONDITIONER for 10 seconds or GC DENTIN CONDITIONER for 20 seconds.

c) Rinse thoroughly with water. Dry by blotting with a cotton pellet or gently blowing with an air syringe (Fig. C-1). DO NOT DESICATE. Best results are obtained when prepared surfaces appear moist (glistening).

2. Mixing:

See directions in A (above), Section 2.

D. TEMPORARY FILLING OF ENDODONTIC ACCESS

1. Cleaning the pulp chamber.

a) After appropriate pulp treatment, clean and gently dry the pulp chamber with an air syringe (Fig. D-1).

b) Fill the chamber with a cotton pellet (Fig. D-2).

2. Mixing:

See directions in A, Section 3, d and e.

E. REMOVAL

1. Care is removed.

a) Remove caries and loose debris with hand instruments.

b) For better retention, it is recommended to gently clean the carious surface with GC CAVITY CONDITIONER for 10 seconds or GC DENTIN CONDITIONER for 20 seconds.

c) Rinse thoroughly with water. Dry by blotting with a cotton pellet or gently blowing with an air syringe (Fig. E-1). DO NOT DESICATE. Best results are obtained when prepared surfaces appear moist (glistening).

2. Mixing:

See directions in A (above), Section 2.

F. CLEANING

1. Care is removed.

a) Remove caries and loose debris with hand instruments.

b) For better retention, it is recommended to gently clean the carious surface with GC CAVITY CONDITIONER for 10 seconds or GC DENTIN CONDITIONER for 20 seconds.

c) Rinse thoroughly with water. Dry by blotting with a cotton pellet or gently blowing with an air syringe (Fig. F-1). DO NOT DESICATE. Best results are obtained when prepared surfaces appear moist (glistening).

3. Mixing:

See directions in A (above), Section 2.

G. HYPERSENSITIVITY

1. Care is removed.

a) Remove caries and loose debris with hand instruments.

b) For better retention, it is recommended to gently clean the carious surface with GC CAVITY CONDITIONER for 10 seconds or GC DENTIN CONDITIONER for 20 seconds.

c) Rinse thoroughly with water. Dry by blotting with a cotton pellet or gently blowing with an air syringe (Fig. G-1). DO NOT DESICATE. Best results are obtained when prepared surfaces appear moist (glistening).

4. Mixing:

See directions in A (above), Section 2.

H. POLYMERISATION

1. Care is removed.

a) Remove caries and loose debris with hand instruments.

b) For better retention, it is recommended to gently clean the carious surface with GC CAVITY CONDITIONER for 10 seconds or GC DENTIN CONDITIONER for 20 seconds.

c) Rinse thoroughly with water. Dry by blotting with a cotton pellet or gently blowing with an air syringe (Fig. H-1). DO NOT DESICATE. Best results are obtained when prepared surfaces appear moist (glistening).

5. Finishing:

See directions in A (above), Section 2.

I. LIGHT CURE

1. Care is removed.

a) Remove caries and loose debris with hand instruments.

b) For better retention, it is recommended to gently clean the carious surface with GC CAVITY CONDITIONER for 10 seconds or GC DENTIN CONDITIONER for 20 seconds.

c) Rinse thoroughly with water. Dry by blotting with a cotton pellet or gently blowing with an air syringe (Fig. I-1). DO NOT DESICATE. Best results are obtained when prepared surfaces appear moist (glistening).

6. Finishing:

See directions in A (above), Section 2.

J. REMOVAL

1. Care is removed.

a) Remove caries and loose debris with hand instruments.

b) For better retention, it is recommended to gently clean the carious surface with GC CAVITY CONDITIONER for 10 seconds or GC DENTIN CONDITIONER for 20 seconds.

c) Rinse thoroughly with water. Dry by blotting with a cotton pellet or gently blowing with an air syringe (Fig. J-1). DO NOT DESICATE. Best results are obtained when prepared surfaces appear moist (glistening).

2. Finishing:

See directions in A (above), Section 2.

K. CLEANING

1. Care is removed.

a) Remove caries and loose debris with hand instruments.

b) For better retention, it is recommended to gently clean the carious surface with GC CAVITY CONDITIONER for 10 seconds or GC DENTIN CONDITIONER for 20 seconds.

c) Rinse thoroughly with water. Dry by blotting with a cotton pellet or gently blowing with an air syringe (Fig. K-1). DO NOT DESICATE. Best results are obtained when prepared surfaces appear moist (glistening).

3. Finishing:

See directions in A (above), Section 2.

L. POLYMERISATION

1. Care is removed.

a) Remove caries and loose debris with hand instruments.

b) For better retention, it is recommended to gently clean the carious surface with GC CAVITY CONDITIONER for 10 seconds or GC DENTIN CONDITIONER for 20 seconds.

c) Rinse thoroughly with water. Dry by blotting with a cotton pellet or gently blowing with an air syringe (Fig. L-1). DO NOT DESICATE. Best results are obtained when prepared surfaces appear moist (glistening).

4. Finishing:

See directions in A (above), Section 2.

M. HYPERSENSITIVITY

1. Care is removed.

a) Remove caries and loose debris with hand instruments.

b) For better retention, it is recommended to gently clean the carious surface with GC CAVITY CONDITIONER for 10 seconds or GC DENTIN CONDITIONER for 20 seconds.

c) Rinse thoroughly with water. Dry by blotting with a cotton pellet or gently blowing with an air syringe (Fig. M-1). DO NOT DESICATE. Best results are obtained when prepared surfaces appear moist (glistening).

5. Finishing:

See directions in A (above), Section 2.

N. REMOVAL

1. Care is removed.

a) Remove caries and loose debris with hand instruments.

b) For better retention, it is recommended to gently clean the carious surface with GC CAVITY CONDITIONER for 10 seconds or GC DENTIN CONDITIONER for 20 seconds.

c) Rinse thoroughly with water. Dry by blotting with a cotton pellet or gently blowing with an air syringe (Fig. N-1). DO NOT DESICATE. Best results are obtained when prepared surfaces appear moist (glistening).

6. Finishing:

See directions in A (above), Section 2.

O. CLEANING

1. Care is removed.

a) Remove caries and loose debris with hand instruments.

b) For better retention, it is recommended to gently clean the carious surface with GC CAVITY CONDITIONER for 10 seconds or GC DENTIN CONDITIONER for 20 seconds.

c) Rinse thoroughly with water. Dry by blotting with a cotton pellet or gently blowing with an air syringe (Fig. O-1). DO NOT DESICATE. Best results are obtained when prepared surfaces appear moist (glistening).

7. Finishing:

See directions in A (above), Section 2.

P. REMOVAL

1. Care is removed.

a) Remove caries and loose debris with hand instruments.

b) For better retention, it is recommended to gently clean the carious surface with GC CAVITY CONDITIONER for 10 seconds or GC DENTIN CONDITIONER for 20 seconds.

c) Rinse thoroughly with water. Dry by blotting with a cotton pellet or gently blowing with an air syringe (Fig. P-1). DO NOT DESICATE. Best results are obtained when prepared surfaces appear moist (glistening).

8. Finishing:

See directions in A (above), Section 2.

Q. CLEANING

1. Care is removed.

a) Remove caries and loose debris with hand instruments.</p

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