
CONTENTS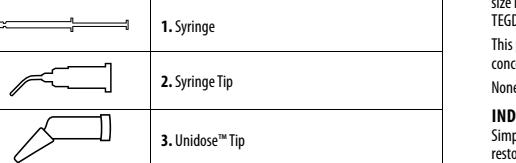

en - ENGLISH

SimpliShade™ Bulk Fill Flow

Nanohybrid Restorative Composite

ISO 4049: Polymer-Based Dental Restorative Material, Class 2/Group 1

GENERAL INFORMATION

SimpliShade™ Bulk Fill Flow is an intra-oral, light-cured, esthetic, resin-based nanohybrid dental restorative that is designed to be used as a base or liner in Class I and II restorations and must be overlaid with a composite resin to replace the occlusal surfaces. It is also suitable to be used on non-occlusal contact surfaces, such as pit and fissures, core, and Class III and V restorations.

SimpliShade Bulk Fill Flow is available in Unidose™ compules and syringe delivery systems that are suitable for occlusal surfaces. The product allows a technique in a cavity 5 mm in depth can be filled and cured in a single increment.

SimpliShade Bulk Fill Flow is offered in one universal shade to optically blend with a range of VITA® shade equivalents. Once light cured, the composite can be finished and polished using conventional finishing and polishing instruments.

When required per the European Medical Device Regulation (MDR EU 2017/745) a Summary of Safety and Clinical Performance document (SSCP) is available and can be obtained at the following website: <https://ec.europa.eu/tools/eudamed>

The Basic UDI-DI for this device is 805151187100011458.

Intended Purpose: The intended purpose of SimpliShade Bulk Fill Flow is for restoration of teeth with small cavities or small core build-ups as a liner/base.

Intended Users: This product is to be used by qualified dental professionals.

Intended Patient Population: The device is for use in a general patient population in need of a restoration due to missing tooth structure.

Clinical Benefit: It restores the anatomy of the tooth and its function.

PRODUCT COMPOSITION

SimpliShade™ Bulk Fill Flow is available in one composite shade and is radiopaque with about 280% aluminum (100% aluminum has a radio-opacity equivalent to that of dentin and 200% aluminum is equivalent to enamel).

The fillers are comprised of chemically infused mixed oxides, silica, and ytterbium trifluoride (approximately 69.7% weight or 47% by volume). The inorganic fillers are

up to 62% by weight or 36% by volume. SimpliShade Bulk Fill Flow has a primary particle size range of 40 nm to 0.1 µm. The product contains Bis-EMA, Bis-GMA, UDMA, and TEGDMA in an approximate concentration of 24% to 40% by weight. This product also contains quinones, phenols, benzoates, and inorganic oxide in a concentration of less than 1% each.

None of the ingredients in the product contain pharmacologically active components.

INDICATIONS FOR USE

SimpliShade Bulk Fill Flow is indicated to be used as a base/liner under all classes of direct restorations, pit and fissure sealant, restoration of small cavities, blocking out undercuts and small core build-ups.

CONTRAINDICATIONS

None known.

WARNINGS

• Do not use this product in patients with known methacrylate allergies. The dental professional should inform the patient that this product is known to cause a methacrylate allergic reaction.

• Avoid product application in close proximity to the pulp tissue, since exposure to this product may injure the pulp.

• Avoid skin, eye, and soft tissue exposure to this product, since uncured methacrylate may cause an irritation reaction. Wash thoroughly with water after contact.

• In case of contact with eyes, rinse immediately with water and seek medical attention.

PRECAUTIONS

• Provide the patient with protective eyewear.

• Use a rubber dam to isolate the oral cavity.

• Avoid fluid contamination during placement.

• Use of protective gloves and a no-touch clinical technique is strongly recommended.

• If the composite is touched by commonly used gloves, acrylicate may penetrate the gloves. Dispose of gloves immediately and wash hands.

• Not to be used in stress bearing and contact surfaces.

• When used as a base or liner, allow at least 2 mm for the placement of the final restorative material.

• Do not use the restorative material on surfaces that are contaminated with hydrogen peroxide (H₂O₂) or eugenol, as its residues can adversely affect the polymerization of the composite.

• Protect the material from ambient light that may initiate polymerization.

• The Unidose™ tips and syringes are designed for SINGLE PATIENT USE ONLY to prevent cross-contamination between patients.

ADVERSE EVENTS

If a serious incident occurs with this medical device, report it to the legal manufacturer, Kerr Italia S.r.l., and to the competent authority for that country in which the user and/or patient is established.

STEP-BY-STEP INSTRUCTIONS

Limitations of Use

SimpliShade™ Bulk Fill Flow is not recommended to be used for large restorations unless it is capped with a conventional restorative composite.

• When a dry working field cannot be established.

• The step-by-step working instructions cannot be applied.

• The restoration cannot be properly polymerized.

Note: Wear personal protective equipment when using and handling this product.

1. Isolate the tooth with your preferred isolation method.

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5. Note: Wear personal protective equipment when using and handling this product.

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+34 968 10 00 00 | kerrdent.com

AUSTRALIA AND NEW ZEALAND
Kerr Australia Pty. Ltd., Unit 1, 12 March Rd,
Lane Cove West, NSW 2066 Australia
+61 2 8700 3000

©2024 Kerr Corporation. All rights reserved.

Kerr

All trademarks are property of Kerr Corporation. All Rights Reserved.

SimpliShade™

Bulk Fill Flow

Nano-hybrid Restorative Composite

Instructions For Use

DMC4363_Rev.00 **2024-01**

UK Kerr UK Ltd.
0066 4 Ormwood Park
4 Ormwood Road, Stockley Park
Uxbridge UB11 0EE, United Kingdom

MD Kerr Mfg Co., Inc.
10000 10th Street, Suite 100
Norcross, GA 30071, USA

CA Kerr Canada Inc.
10000 10th Street, Suite 100
Norcross, GA 30071, USA

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Kerr, SimpliShade, OptiBond, und Unidose are trademarks owned by Kerr Corporation. ©2024 Kerr Corporation. All trademarks are property of Kerr Corporation. All Rights Reserved. VITA is a trademark of VITA Zahnfabrik Company, Germany.
DK Kerr Deutschland GmbH