
## CONTENTS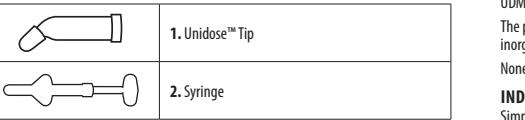

### en - ENGLISH

#### SimpliShade™ Bulk Fill

Nano-hybrid Restorative Composite  
ISO 4049: Polymer-Based Dental Restorative Material, Class 2/Group 1

#### GENERAL INFORMATION

SimpliShade Bulk Fill is an intra-oral, light-cured, esthetic, resin-based nano-hybrid dental restorative that is designed for the placement of direct restorations. SimpliShade Bulk Fill is designed to be used after placement of a dental adhesive, such as a product from the Kerr OptiBond® family.

SimpliShade Bulk Fill is available in Unidose™ compules and syringe delivery systems that are suitable for occlusal surfaces. The product allows a technique in which a cavity up to 5 mm in depth can be filled and cured in a single increment.

SimpliShade Bulk Fill is offered in one universal shade to optically blend with a range of VITA™ shade guide equivalents.

This composite can be polished with a wide variety of polishing discs, rubber cups, and brushes.

When required per the European Medical Device Regulation (MDR EU 2017/745) a Summary of Safety and Clinical Performance Document (SSCP) is available and can be obtained at the following website: <https://ec.europa.eu/tools/eudamed>. The Basic UDI-DI for this device is 80515118710001458.

**Intended Purpose:** The intended purpose of SimpliShade Bulk Fill is for restoration of teeth or as a core build-up material.

**Intended users:** This product is to be used by qualified dental professionals.

**Intended Patient Population:** The device is for use in a general patient population in need of a restoration due to missing tooth structure.

**Clinical Benefit:** It restores the anatomy of the tooth and its function.

#### PRODUCT COMPOSITION

SimpliShade Bulk Fill composite is radiopaque with about 25% aluminum (100% aluminum has a radio-opacity equivalent to that of dentin and 200% aluminum is equivalent to enamel).

The fillers are comprised of chemically infused mixed oxides, pre-polymerized filler, barium glass filler, silica, and ytterbium trifluoride (total filler loading of approximately 82.5% weight or 66% by volume).

The inorganic filler content is up to 75% weight or 55% by volume. SimpliShade Bulk Fill

has a primary particle size range of 40 nm to 0.4 µm. The product contains Bis-EMA, Bis-GMA, UDMA, and TEGDMA resins in an approximate concentration of 14% to 37% by weight. The product also contains quinones, phenols, polyesters, polyamides, benzoates, and inorganic oxides in a concentration of less than 1% each.

None of the ingredients in the product contain pharmacologically active components.

#### INDICATIONS FOR USE

SimpliShade Bulk Fill is indicated for direct placement of anterior and posterior restorations, core build-ups, and repair of defects in restorations.

#### CONTRAINDICATIONS

None known.

#### WARNINGS

• Do not use this product in patients with known methacrylate allergies. The dental professional should inform the patient that this product is known to cause a methacrylate allergic reaction.

• Avoid product application in close proximity to the pulp tissue, since exposure to this product may injure the pulp.

• Avoid skin, eye, and soft tissue exposure to this product, since uncured methacrylate may cause an irritation reaction. Wash thoroughly with water after contact.

• In case of contact with eyes, rinse immediately with water and seek medical attention.

• Do not take internally.

#### PRECAUTIONS

• Provide the patient with protective eyewear.

• Use a rubber dam to isolate the oral cavity.

• Avoid contamination of the preparation with fluids during restorative placement.

• Use of protective gloves and a no-touch clinical technique is strongly recommended.

• If the composite is touched with commonly used gloves, acrylate may penetrate the gloves. Dispose of gloves immediately and wash hands.

EXCEPT AS EXPRESSLY PROVIDED ABOVE, THERE ARE NO WARRANTIES, BY KERR ITALIA, EXPRESS OR IMPLIED, INCLUDING WARRANTIES WITH RESPECT TO DESCRIPTION, QUALITY, OR FITNESS FOR A PARTICULAR PURPOSE.

Kerr, SimpliShade, OptiBond, and Unidose are trademarks owned by Kerr Corporation. ©2024 Kerr Corporation. All trademarks are property of Kerr Corporation. All Rights Reserved. VITA is a trademark of VITA Zahnfabrik Company, Germany.

Full explanation of symbols used on Kerr packaging is located at: <http://www.keridental.com/symbols-glossary>

#### CLEANING AND DISINFECTION

**Unidose tips:** To prevent the cross-contamination between patients, do not re-use Unidose tips.

The product also contains quinones, phenols, polyesters, polyamides, benzoates, and inorganic oxides in a concentration of less than 1% each.

None of the ingredients in the product contain pharmacologically active components.

#### INDICATIONS FOR USE

SimpliShade Bulk Fill is indicated for direct placement of anterior and posterior restorations, core build-ups, and repair of defects in restorations.

#### CONTRAINDICATIONS

None known.

#### SHELF LIFE, STORAGE AND DISPOSAL

SimpliShade Bulk Fill should be stored at temperatures between 2°C to 28°C (36°F to 82°F).

Avoid excess heat. If the product is stored in a refrigerator, allow the product to reach room temperature prior to use.

The expiration date is marked on the packaging. Do not use the restorative after the expiration date. Once the foil pouch is opened, use within 6 months and prior to the expiration date printed on the label.

For proper disposal always follow federal or state laws applicable to that country.

#### LIMITED WARRANTY - LIMITATION OF KERR ITALIA'S LIABILITY

Kerr Italia's technical advice, whether verbal or in writing, is designed to assist dentists in using Kerr Italia's product. The dentist assumes all risk and liability for damages arising out of the improper use of Kerr Italia's product. In the event of a defect in material or workmanship, Kerr Italia's liability is limited, at Kerr Italia's option, to replacement of the defective product or, if appropriate, reimbursement of the actual cost of the defective product. In order to take advantage of this limited warranty, the defective product must be returned to Kerr Italia. In no event shall Kerr Italia be liable for any indirect, incidental, or consequential damages.

In case of contact with eyes, rinse immediately with water and seek medical attention.

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©2024 Kerr Corporation.  
All trademarks are property of Kerr Corporation. All Rights Reserved.

**Kerr**

Bulk Fill

SimpliShade™

Bulk Fill

Nanohybrid Restorative Composite

DMC4362\_Rev.00

UK Authorised Rep in UK:  
Kerr Ltd.  
0086 0818 700 0000  
0086 0818 700 0000  
Kerr Australia Pty. Ltd. Unit 12, 12 Mars Rd,  
Lane Cove West, NSW 2066, Australia  
+61 2 8700 3000

©2024 Kerr Corporation.  
All trademarks are property of Kerr Corporation. All Rights Reserved.

**Kerr**

Bulk Fill

SimpliShade™

Bulk Fill

Nanohybrid Restorative Composite

Instructions For Use

DMC4362\_Rev.00

UK Authorised Rep in UK:  
Kerr Ltd.  
0086 0818 700 0000  
0086 0818 700 0000  
Kerr Australia Pty. Ltd. Unit 12, 12 Mars Rd,  
Lane Cove West, NSW 2066, Australia  
+61 2 8700 3000

©2024 Kerr Corporation.  
All trademarks are property of Kerr Corporation. All Rights Reserved.

**Kerr**

Bulk Fill

SimpliShade™

Bulk Fill

Nanohybrid Restorative Composite

DMC4362\_Rev.00

UK Authorised Rep in UK:  
Kerr Ltd.  
0086 0818 700 0000  
0086 0818 700 0000  
Kerr Australia Pty. Ltd. Unit 12, 12 Mars Rd,  
Lane Cove West, NSW 2066, Australia  
+61 2 8700 3000

©2024 Kerr Corporation.  
All trademarks are property of Kerr Corporation. All Rights Reserved.

**Kerr**

Bulk Fill

SimpliShade™

Bulk Fill

Nanohybrid Restorative Composite

DMC4362\_Rev.00

UK Authorised Rep in UK:  
Kerr Ltd.  
0086 0818 700 0000  
0086 0818 700 0000  
Kerr Australia Pty. Ltd. Unit 12, 12 Mars Rd,  
Lane Cove West, NSW 2066, Australia  
+61 2 8700 3000

©2024 Kerr Corporation.  
All trademarks are property of Kerr Corporation. All Rights Reserved.

**Kerr**

Bulk Fill

SimpliShade™

Bulk Fill

Nanohybrid Restorative Composite

DMC4362\_Rev.00

UK Authorised Rep in UK:  
Kerr Ltd.  
0086 0818 700 0000  
0086 0818 700 0000  
Kerr Australia Pty. Ltd. Unit 12, 12 Mars Rd,  
Lane Cove West, NSW 2066, Australia  
+61 2 8700 3000

©2024 Kerr Corporation.  
All trademarks are property of Kerr Corporation. All Rights Reserved.

**Kerr**

Bulk Fill

SimpliShade™

Bulk Fill

Nanohybrid Restorative Composite

DMC4362\_Rev.00

UK Authorised Rep in UK:  
Kerr Ltd.  
0086 0818 700 0000  
0086 0818 700 0000  
Kerr Australia Pty. Ltd. Unit 12, 12 Mars Rd,  
Lane Cove West, NSW 2066, Australia  
+61 2 8700 3000

©2024 Kerr Corporation.  
All trademarks are property of Kerr Corporation. All Rights Reserved.

**Kerr**

Bulk Fill

SimpliShade™

Bulk Fill

Nanohybrid Restorative Composite

DMC4362\_Rev.00

UK Authorised Rep in UK:  
Kerr Ltd.  
0086 0818 700 0000  
0086 0818 700 0000  
Kerr Australia Pty. Ltd. Unit 12, 12 Mars Rd,  
Lane Cove West, NSW 2066, Australia  
+61 2 8700 3000

©2024 Kerr Corporation.  
All trademarks are property of Kerr Corporation. All Rights Reserved.

**Kerr**

Bulk Fill

SimpliShade™

Bulk Fill

Nanohybrid Restorative Composite

DMC4362\_Rev.00

UK Authorised Rep in UK:  
Kerr Ltd.  
0086 0818 700 0000  
0086 0818 700 0000  
Kerr Australia Pty. Ltd. Unit 12, 12 Mars Rd,  
Lane Cove West, NSW 2066, Australia  
+61 2 8700 3000

©2024 Kerr Corporation.  
All trademarks are property of Kerr Corporation. All Rights Reserved.

**Kerr**

Bulk Fill

SimpliShade™

Bulk Fill

Nanohybrid Restorative Composite

DMC4362\_Rev.00

UK Authorised Rep in UK:  
Kerr Ltd.  
0086 0818 700 0000  
0086 0818 700 0000  
Kerr Australia Pty. Ltd. Unit 12, 12 Mars Rd,  
Lane Cove West, NSW 2066, Australia  
+61 2 8700 3000

©2024 Kerr Corporation.  
All trademarks are property of Kerr Corporation. All Rights Reserved.

**Kerr**

Bulk Fill

SimpliShade™

Bulk Fill

Nanohybrid Restorative Composite

DMC4362\_Rev.00

UK Authorised Rep in UK:  
Kerr Ltd.  
0086 0818 700 0000  
0086 0818 700 0000  
Kerr Australia Pty. Ltd. Unit 12, 12 Mars Rd,  
Lane Cove West, NSW 2066, Australia  
+61 2 8700 3000

©2024 Kerr Corporation.  
All trademarks are property of Kerr Corporation. All Rights Reserved.

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Kerr, SimpliShade, OptiBond, und Unidose are trademarks owned by Kerr Corporation.
SimpliShade Kütleşel Dolgu, çok çeşitli işkla, nanohibrit bir dental restoratif materyaldir. Kerr, SimpliShade, OptiBond, and Unidose are trademarks owned by Kerr Corporation. ©2024 Kerr Corporation. All trademarks are property of Kerr Corporation. All Rights Reserved. VITA is a trademark of VITA Zahnfabrik Company, Germany.