ENGLISH
<p>TAKE 1™ ADVANCED™ BITE</p> REGISTRATION MATERIAL
DIRECTIONS FOR USE
<p>Take 1™ Advanced™ Bite Registration is an addition-cure vinyl polysiloxane material designed for making accurate interocclusal records.</p> <p>Inserting Cartridge into the Extruder</p> <ol style="list-style-type: none">Push metal lever at back of extruder up (toward plunger assembly) and pull back on plunger assembly. Lift up on lock assembly on top of extruder exposing slot for insertion of cartridge. Insert cartridge with notch in collar of cartridge down to match protrusion of extruder. Snap lock assembly down locking cartridge into place.
Attaching Mixing Tip to Cartridge
<ol style="list-style-type: none">While holding cartridge with cap toward you, rotate cap 90° counter clockwise, moving alignment tab on cap to align with alignment notch on cartridge. Separate cap from cartridge by pulling cap off. Secure mixing tip and position alignment tab with notch on cartridge. Push mixing tip down onto cartridge. Rotate cartridge 90° clockwise. This will lock mixing tip on cartridge.
Extruder Operation
<ol style="list-style-type: none">Squeeze trigger with moderate and even pressure to begin mixing material. Material will stop flowing when trigger is released.
Extruder Cleaning
<ol style="list-style-type: none">Extruder may be cleaned with commercially available detergents. Do not use organic solvents to clean extruder. Solvents may dissolve and damage components.

Take 1 Advanced Bite Registration General Timelines
Working Time: 15 seconds from start of mixing.
Mouth Removal Time: 1 minute from start of mixing.

- Work time is based on material at 23°C. Set time is based on material at 37°C. These times may vary based on storage conditions, temperature, humidity, and age of product.

Take 1 Advanced Bite Registration Placement

Dispense Take 1 Advanced Bite Registration directly from the mixing tip onto the occlusal or incisal surfaces of the teeth to be registered to a depth of about 5 mm.

To record anterior teeth, extrude Take 1 Advanced Bite Registration onto both the mandibular and maxillary surfaces to a depth of about 5 mm.

For use with a bite registration tray, dispense Take 1 Advanced Bite Registration on both sides of the tray to a depth of 2–3 mm.

Take 1 Advanced Bite Registration Removal
Remove Take 1 Advanced Bite Registration from the mouth at 1 minute from start of mixing by gently pulling the material from the teeth.

Model Placement

Take 1 Advanced Bite Registration is easily carved to remove excess material which would impede the placement on the model. Take 1 Advanced Bite Registration achieves maximum rigidity at 3 minutes from start of mix, allowing immediate model placement into bite registration.

ALTERNATE USES

Check Fit Of Castings

Take 1 Advanced Bite Registration is also indicated for checking the fit of castings. Line the casting with a thin layer of material and seat. Take 1 Advanced Bite Registration’s fast set time allows a much faster procedure and the color contrast to the casting provides visual confirmation of impinged areas.

Implant Transfer Coping Impressions

The high rigidity of Take 1 Advanced Bite Registration provides an ideal implant transfer coping impression material. Take 1 Advanced Bite Registration minimizes mouth removal distortion and is easily removed from the mouth without risk of fracture or patient trauma. All undercut should be blocked out due to the rigid set of the material.

STORAGE
Store at ambient temperature. Avoid prolonged exposure to excessive heat.

CAUTIONS

- Use of products containing active sulfur, aluminum chloride or nitrogen compounds (retraction cords containing ferric sulfate, polysulfide impression materials, latex gloves, etc.) in conjunction with this product will interfere with the setting reaction of the Polyvinylsiloxane material. Use of these materials requires the area to be rinsed thoroughly to remove all residues. Handling retraction cord with latex gloves may prevent setting of the impression material in the area of glove contact with the retraction cord.
- If gloves are worn when handling this material, test the gloves in advance for compatibility. Certain gloves will inhibit the setting of vinylpolysiloxane materials.
- Avoid splattering the material on clothing as it cannot be removed by dry cleaning.

- Do not use material after expiry date.
- Do not use with polyether materials.

Limited Warranty - Limitation of Kerr's Liability

Kerr's technical advice, whether verbal or in writing, is designed to assist dentists in using Kerr's product. The dentist assumes all risk and liability for damages arising out of the improper use of Kerr's product. In the event of a defect in material or workmanship, Kerr's liability is limited, at Kerr's option, to replacement of the defective product or part thereof, or reimbursement of the actual cost of the defective product. In order to take advantage of this limited warranty, the defective product must be returned to Kerr. In no event shall Kerr be liable for any indirect, incidental, or consequential damages. EXCEPT AS EXPRESSLY PROVIDED ABOVE, THERE ARE NO WARRANTIES, BY KERR, EXPRESS OR IMPLIED, INCLUDING WARRANTIES WITH RESPECT TO DESCRIPTION, QUALITY, OR FITNESS FOR A PARTICULAR PURPOSE.

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